# WatchBP® 03

Instruction Manual  
Bedienungsanleitung  
Mode d'emploi  
Manuale di istruzioni

**EN** → 3

**DE** → 41

**FR** → 79

**IT** → 117

*microlife*

# WatchBP<sup>®</sup> 03

**Affordable ABPM and strictly guideline-based SBPM for accurate out-of-office blood pressure measurement.**

**Instruction Manual**

The Microlife WatchBP O3 (Out Of Office) is designed to provide reliable and unbiased ambulatory and self blood pressure measurement results, and strictly follows European Society of Hypertension (ESH) and American Heart Association (AHA) recommendations for out-of-office blood pressure measurement <sup>1</sup>.The WatchBP O3 device has been clinically validated according to the ESH protocol.

<sup>1</sup> O'Brien E, Asmar R, Beilin L, Imai Y, et al. European Society of Hypertension recommendations for conventional, ambulatory and home blood pressure measurement. European Society of Hypertension Working Group on Blood Pressure Monitoring. *J Hypertens* 2003;21:821-848.

# Table of Contents

## Product description

Name of parts and display ..... 3-4

## Before using WatchBP 03 for the first time

Activating the device ..... 5

Confirm cuff size ..... 6

## Selecting the operation mode

«AMBULATORY» Mode ..... 7-9

«HOME» Mode..... 9-11

«CASUAL» Mode ..... 12

## Recording the taking of medication

Medication record ..... 13-14

## Measurement interval programming

Installation of the software program..... 15

Connecting the device to computer ..... 15

Programming the interval schedule ..... 16

## Taking blood pressure measurements

«AMBULATORY» mode ..... 17-21

«HOME» mode ..... 22-23

«CASUAL» mode..... 24-25

## Viewing, deleting and transferring measurements

Viewing measurements..... 26-28

Deleting measurements ..... 29

Transferring measurements ..... 30

## Appendix

Batteries and power adaptor ..... 31

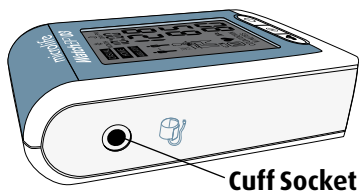
Safety, care, accuracy test and disposal..... 32-33

Error messages..... 34-35


Technical specifications ..... 36

# Product description

## Name of parts


Cuff Socket

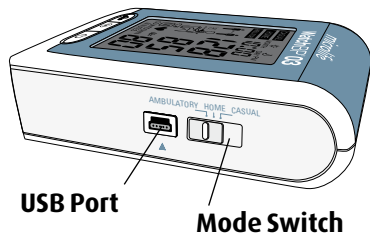
USB Port

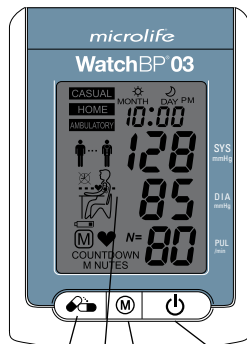Mode Switch


Display  
Medication Button

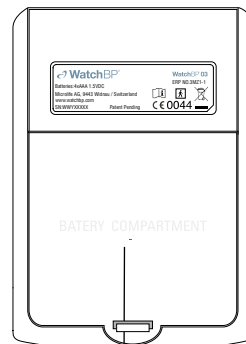ON/OFF Button  
M Button (Memory)

Battery Compartment


# Display


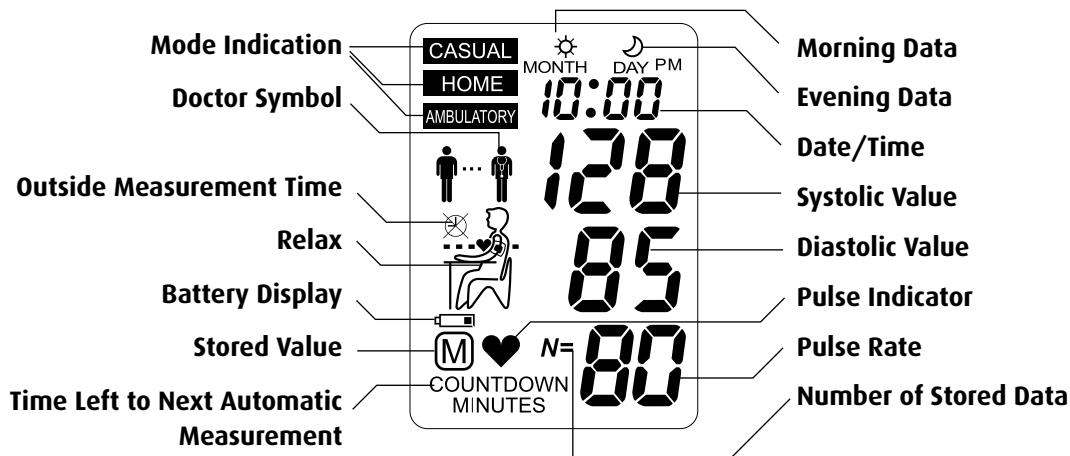
# Before using WatchBP 03 for the first time

## Activating the Device

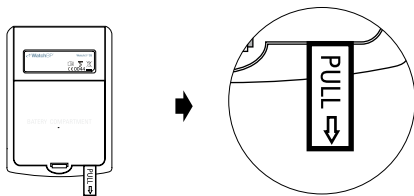Pull out the protective strip from the battery compartment.

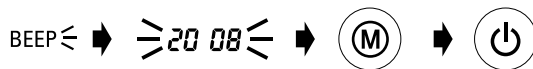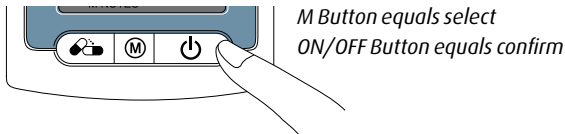- 1) **Set the year** – Upon removing the protective strip or installing new batteries, the Year number flashes in the display. Use the M Button to select the Year. Press the ON/OFF Button to confirm your selection.

- 2) **Set the month** – Use M Button to set the Month. Press the ON/OFF Button to confirm.

- 3) **Set the day** – Press M Button to set the Day. Press the ON/OFF Button to confirm.

- 4) **Set the time** – Once the Hour and Minutes have been set and the ON/OFF Button pressed, the date and time are set, and the current time is displayed.

- 5) To change the date and time, briefly take out and put back one battery from the battery compartment. The Year number will flash. Complete the process as described above.

## Confirm cuff size

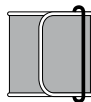
A variety of different cuff sizes are available for use with the WatchBP 03 device.

*\* please use only Microlife cuffs!*


### M (Medium size)

22 - 32 cm (8.7 - 12.6 inches)  
*M is the correct size for most people.*



### L (Large size)

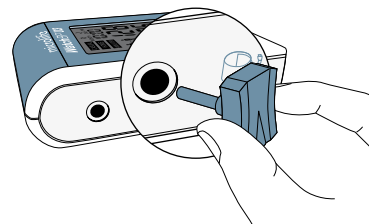32-42 cm (12.6-16.5 inches)


# Selecting the operation mode

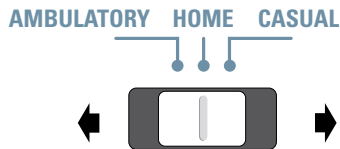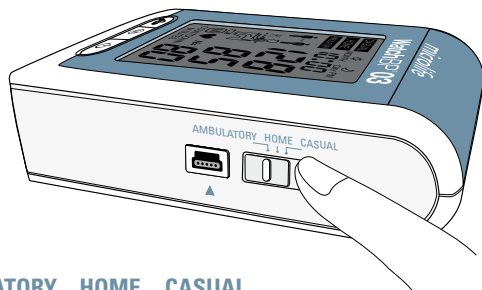Prior to each measurement, use the Mode Switch on the side of the device to select the proper measurement mode. The WatchBP O3 offers three measurement modes: «**AMBULATORY**», «**HOME**» or «**CASUAL**» mode.

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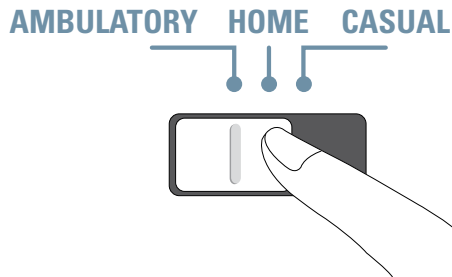Select «**AMBULATORY**» mode for fully programmable 24-hour patient out-of-office blood pressure measurement.

## Programmable measurement intervals

The device automatically takes measurements at fixed intervals of 15, 20, 30, or 60 minutes, as programmed by the physician.

**15** | **30**  
**20** | **60**    **minutes**

❖ *The default measurement interval is 30 minutes for the awake hours and 60 minutes for the asleep hours.*

## Set awake and asleep hours

To better suit each patient's lifestyle, awake and asleep hours can be programmed using the provided WatchBP 03 software. Intervals for the awake and asleep hours can also be programmed independently.

— Setting Ambulatory Measurement Schedule —

Awake from 06 AM ~ 10 PM with 30 minutes interval

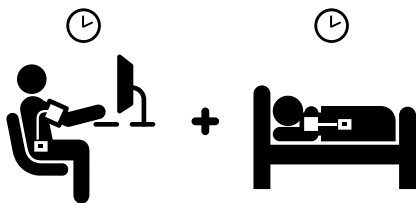
Asleep from 10 PM ~ 06 AM with 60 minutes interval

## Selecting the operation mode (cont.)

### Two measurement periods

The awake measurement period can be programmed to start at anytime between 4AM to 10AM, and the asleep measurement period can be set to anytime between 9PM to 2AM. Measurement intervals can be set to 15, 20, 30, or 60 minutes for both awake and asleep time periods.


### «HOME» Mode

Select «**HOME**» mode for patient home blood pressure measurement in accordance with the guidelines of the European Society of Hypertension (ESH) and American Heart Association (AHA).

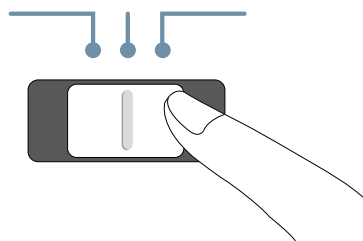AMBULATORY HOME CASUAL


### No measurements on non-work days

The patient should carry out readings on 7 consecutive working days (or normal week days),

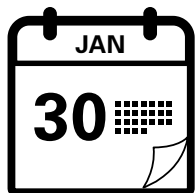**No readings should be taken on «non-working» days** (or particularly relaxing days) in this mode!

**7** working days

### Two sets of measurements per day

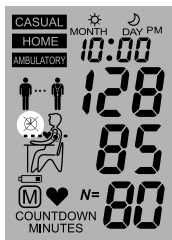
ESH guidelines recommend one double measurement be taken in the morning between 06:00 - 09:00 and one double measurement taken in the evening between 18:00 - 21:00.


## Selecting the operation mode (cont.)

### Extended measurement period

The WatchBP O3 device offers an extended measurement period and allows morning measurements between 04:00 - 12:00 and evening measurements between 18:00 - 24:00.

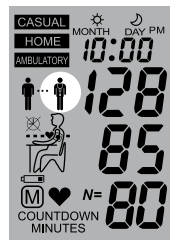Outside these times, measurements cannot be taken and the symbol on the left will be displayed on the screen.


When measurements have been carried out for the full 7 days, the doctor symbol will flash on the screen.

## «CASUAL» Mode

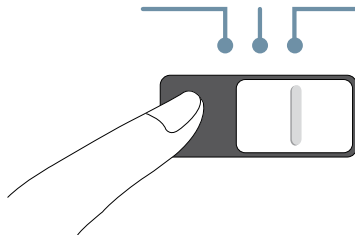In «CASUAL» mode, the device functions like a regular blood pressure monitor – single measurements are automatically stored and can be reviewed by the physician at a later date.


AMBULATORY HOME CASUAL

## 250 measurements safely stored

The WatchBP 03 device can store up to 250 single measurements in «CASUAL» mode.

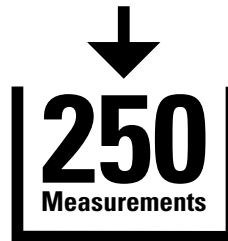
*\* When memory is full, each new measurement will automatically overwrite the earliest measurement.*

# Recording the taking of medication

## Medication record

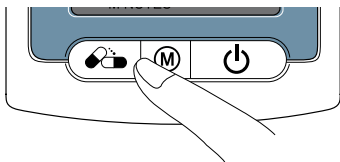The patient can record the time medication is taken by pressing the Medication Button.



- 1) Press and hold the Medication Button for 2 seconds and the Pill icon will appear.

- 2) Release the Medication Button and the Pill icon will flash in alternation with the recorded time.

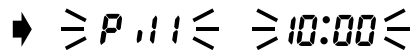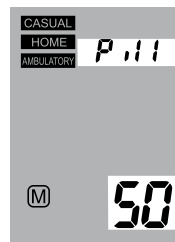➔ **Press and hold**

- 3) Medication record is saved after the beep sound.

**BEEP** ⇐ ➔ **saved**

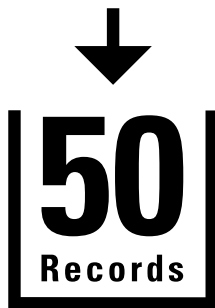
\* A Medication record can be recorded in any of the three modes.

\* The beeper can be disabled via software setting.

## 50 records safely stored


The WatchBP 03 device can store up to 50 medication records.

\* When the memory is full, the device will display Full.

## Delete stored medication records

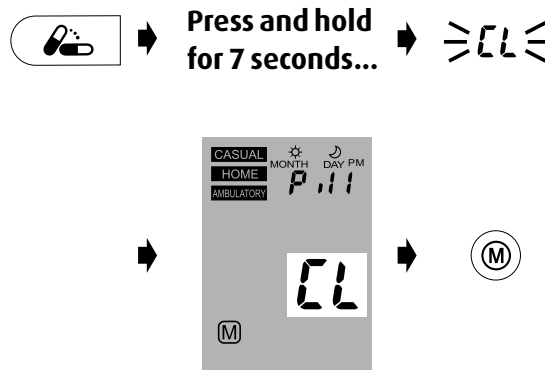To clear the memory of all medication records, press and hold the Medication Button for 7 seconds. The "CL" symbol will flash. Press M to delete the memory or ON/OFF to cancel the deletion .


\* Pressing the M Button to delete data will erase all medication record data in the selected mode.

# Measurement interval programming

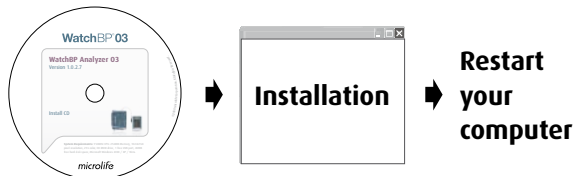
## Installing the software program

- 1) Put the CD in the CD-ROM drive of your computer.  
Alternatively click on «**setup.exe**» in the CD's directory.
- 2) Simply follow the instructions provide in the installation window on the computer screen.
- 3) When installation is finished, be sure to restart the computer before working with the program for the first time.

## Connecting the device to a computer

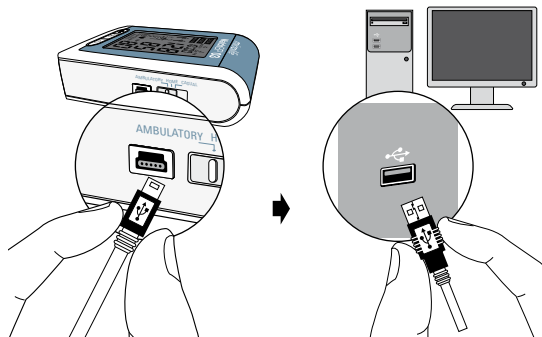- 1) Connect the device to computer. A successful connection is displayed by «**PC**» on the device.
- 2) Start the software program.
- 3) Enter name, identity number, and date of birth to create a new record (if required).

## Programming the interval schedule

- 1) Using the drop-down menu on the lower-left of the screen, choose from 4AM to 10AM as the start of the awake hours.

**Awake from**  ▼ ~

- 2) Using the drop-down menu on the lower-left of the screen, choose from 9 PM to 2 AM as the start of the asleep hours.

**Asleep from**  ▼ ~

- 3) Assign the measurement intervals for the awake and asleep hours by using the drop-down menu to choose from 15, 20, 30, and 60 minutes as the measurement interval.

15	▼
20	
30	
60	

**minutes interval**

15		30
20		60

**minutes**

- 4) Once you have completed the settings, press "Program to BP device" to program the schedule into the device.

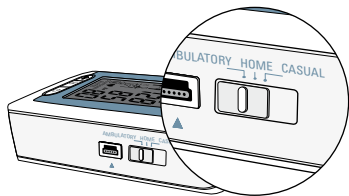
**Program to BP device**


# Taking blood pressure measurements

## In «AMBULATORY» mode

Be sure the device is set to «**AMBULATORY**» Mode.

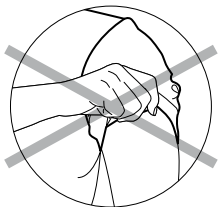
- 1) **Preparing measurement arm** – Remove all clothing covering or constricting the measurement arm.


❖ *Avoid rolling up long sleeves, as this may lead to constriction of blood flow to the measurement arm.*

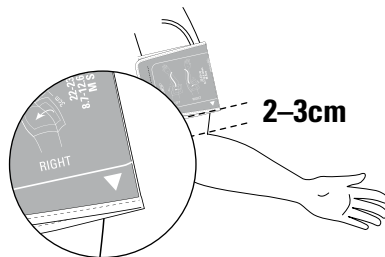- 2) **Positioning the cuff and device** – Fit the cuff closely, but not too tight. Make sure that the cuff is 2~3 cm (1 inch) above the elbow with the tube on the inside of the arm. The measurement tube should point upward and be mounted over the patient's shoulder.

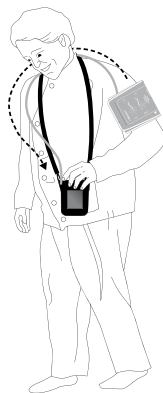
❖ *Adjust the tube holder on the belt of the pouch so it is properly positioned on the shoulder. Attach the tube to the holder so the tube goes up over the shoulder to the other side of the body. An alternative method is to use a waist belt.*


### 3) Proper Positioning the cuff using Anchor Strap

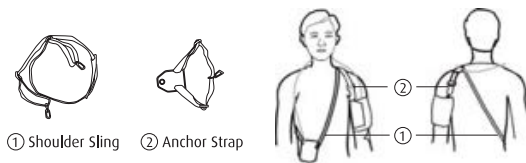**Option 1:** Use the Shoulder Sling ① and Anchor Strap ② to position the cuff as demonstrated in the diagram below.

Instructions: Place the Shoulder Sling over the left shoulder. Insert the device into the holster. Position the device at front. Apply the Cuff to the arm. Buckle up Anchor Strap with Shoulder Sling. Snap the connector of Anchor Strap into D-ring of cuff. Adjust length for fit and comfort.

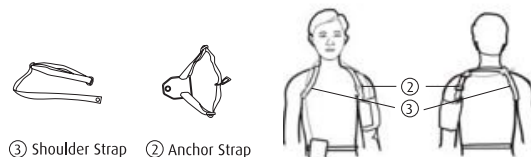**Option 2:** Use the Shoulder Strap ③ and Anchor Strap ② to position the cuff as demonstrated.



Instructions: Place the Shoulder Strap over both shoulders as demonstrated in the diagram. Connect the device to a belt. Position the device at front. Apply the Cuff to the arm. Buckle up Anchor Strap with Shoulder Strap. Snap the connector of Anchor Strap into D-ring of cuff. Adjust length for fit and comfort.

- 4) **Next measurement indicator** – The device will display the next measurement time, indicated by a countdown in minutes on the display.

❖ *The screen will display X countdown minutes to indicate the time to the next automatic measurement.*

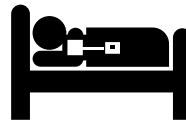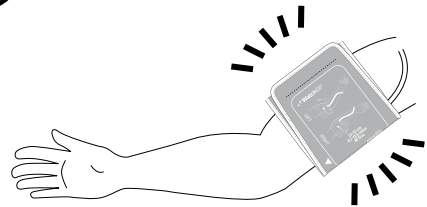- 5) **Measurement reminder** – One minute before the next scheduled measurement, the device will partially inflate and immediately deflate the cuff to remind the patient of the next measurement.


❖ *Measurement reminders will be disabled for the asleep hours.*

## Taking blood pressure measurements (cont.)

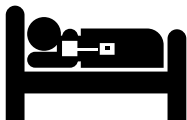- 6) **Measurement reminder** – Five seconds before the next measurement, the device will emit a short series of beeps to notify the patient of the coming measurement.
- 7) **During measurements** – the patient should be reminded to remain still, refrain from talking, and to breath normally during the measurement. If driving or operating equipment when a measurement begins, the patient should, when safe, relax the measurement arm.

※ The beeper of Ambulatory Mode can be disabled via software setting.

※ The device will not beep prior to asleep measurement in «AMBULATORY» mode.

※ The patient can stop any single measurement at anytime by pressing the ON/OFF Button.

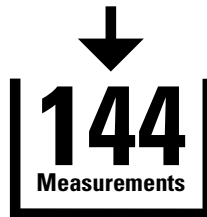
- 8) **Repeat measurements in the case of an error** – The device will automatically repeat the measurement after a two minute countdown should an error occur during measurement.

➔ **A repeat measurement is performed automatically if an error occurs.**

❖ *If the repeat measurement encounters an error, the device will stop taking measurements. The patient should return the device to the physician to determine the cause of error.*

- 9) **Storing measurement data** – The WatchBP O3 automatically stores each measurement result, along with the measurement date and time. A total of 144 measurements can be stored in «**AMBULATORY**» mode.


❖ *The display of BP reading and memory of Ambulatory Mode can be disabled via software setting.*

## Taking blood pressure measurements (cont.)
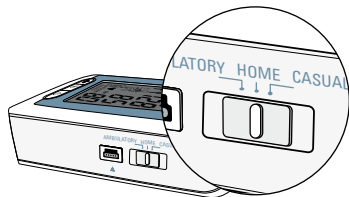
### In «HOME» mode

Be sure the device is set to «HOME» Mode.


- 1) **Before the measurement** – Patients should be informed to avoid physical activity, eating or smoking before each measurement. The patient should sit down for at least 5 minutes before the measurement and relax.


Relax 5 mins

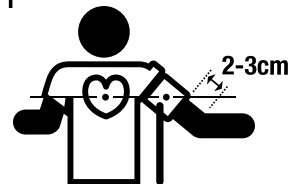- 2) **Applying the cuff** – Patients should be trained on how to properly fit the cuff to their own arm. The cuff should fit closely around the arm, but not too tight, and be placed 2~3cm above the elbow with the tube on the inside of the arm. The patient should support the arm at the same height as the heart during measurement.


※ Additional instructions on taking proper blood pressure measurement can be found on the [www.watchbp.com](http://www.watchbp.com) website.

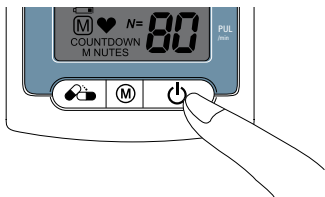
- 3) **Start the measurement** – Press the ON/OFF Button to start measurement.

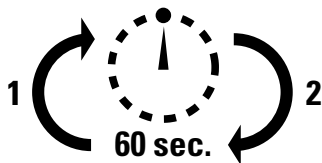- 4) **During the measurement** –The cuff will inflate automatically. One measurement cycle includes two measurements. Once the first measurement is complete, the second measurement will begin after 60-seconds.

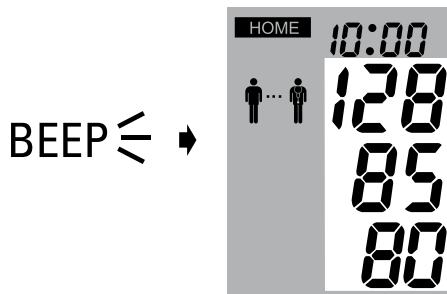
- 5) **Finishing the measurement** – Once the two measurements are complete, measurement data is automatically stored for later reference by the physician. If an error displays after the measurements, the process should be repeated once again.

## Taking a blood pressure measurements (cont.)

### In «CASUAL» mode

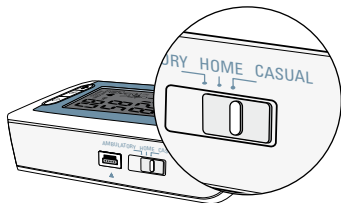
Be sure the device is set to «CASUAL» Mode.

- 1) **Before each measurement** – Patients should be informed to avoid physical activity, eating or smoking before each measurement. The patient should sit down for at least 5 minutes before the measurement and relax.

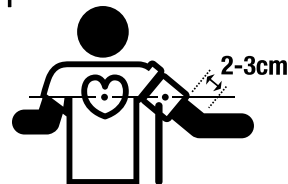
- 2) **Applying the cuff** – Patients should be trained on how to properly fit the cuff to their own arm. The cuff should fit closely around the arm, but not too tight, and be placed 2~3cm above the elbow with the tube on the inside of the arm. The patient should support the arm at the same height as the heart during measurement.


\* Additional instructions on taking proper blood pressure measurement can be found on the [www.watchbp.com](http://www.watchbp.com) website.

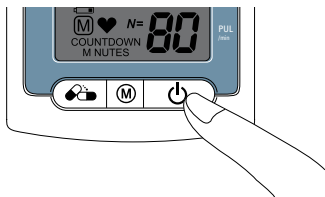- 3) **Start the measurement** – Press the ON/OFF Button to start measurement.

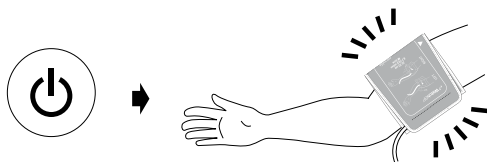
- 4) **During the measurement** – The cuff will inflate automatically. A single measurement will be completed for each push of the ON/OFF Button.

- 5) **Finishing the measurement** – Once the measurement is complete, measurement data is automatically stored for later reference by the physician. If an error displays after the measurements, the process should be repeated once again.

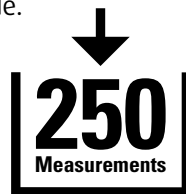- 6) **Storing measurement data** – The WatchBP 03 automatically stores each measurement result, along with the measurement date and time. A total of 250 measurements can be stored in «CASUAL» mode.



# Viewing, deleting and transferring measurements

## Viewing measurements

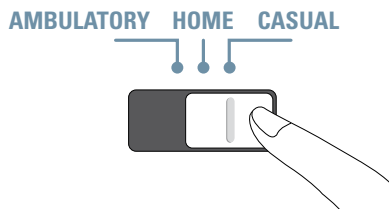- 1) Use the Mode switch to first select the type of measurements wished to be viewed.

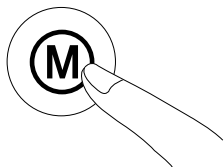- 2) Then press the M Button.

## In «AMBULATORY» mode

- 1) When the M Button is pressed, it briefly displays the total number of measurements stored, e.g. N=20.

\* «A» is displayed when the number shown is the average of all data.

\* «- -» will display when the number of measurements is less than 12.

- 2) Pressing the M Button again displays the average of all awake hours.

- 3) Pressing M Button once again shows the average of all night hours.

- \* All individual readings can be viewed by repeatedly pressing the M Button.
- \* The display of BP reading and memory of Ambulatory Mode can be disabled via software setting.

### In «Home» mode

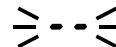
- 1) When the M Button is pressed, it briefly displays the total number of measurements stored, e.g. N=20.

\* «A» is displayed when the number shown is the average of all data.

\* «- -» will display when the number of measurements is less than 12.

## Viewing, deleting and transferring measurements (cont.)

- 2) Pressing the M Button again displays the average of all morning data.

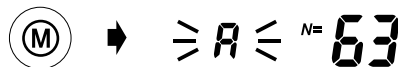
- 3) Pressing M Button once again shows the average of all evening data.

※ All individual readings can be viewed by repeatedly pressing the M Button.

## In «CASUAL» mode

- 1) When the M Button is pressed, it briefly displays the total number of measurements stored, e.g. N=63, followed immediately by the average of all measurements stored in memory.


- 2) All individual readings can be viewed by repeatedly pressing the M Button.

## Deleting measurements

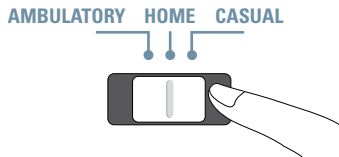Data from «**AMBULATORY**», «**HOME**» and «**CASUAL**» mode measurements can be deleted independent of each other.

- 1) Use the Mode switch to first select the type of measurements you wish to delete.

- 2) Press the M Button and hold it until the CL symbol flashes.

- 3) Release the M Button and press it once more while the delete symbol flashes.


※ Each deletion will erase entire records of each mode.

## Viewing, deleting and transferring measurements (cont.)
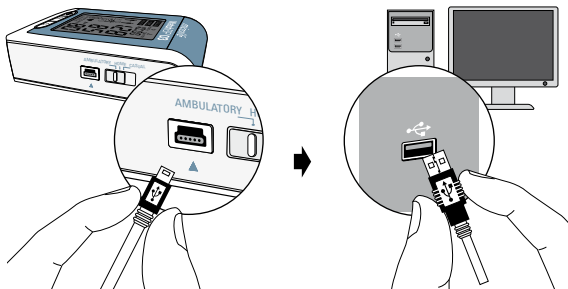
### Transferring measurements

- 1) Connect the device to computer. A successful connection is displayed by «PC» on the device.
- 2) Start the software program.
- 3) To transfer the data in all three modes and the medication compliance record, click «**Download BP data to PC**».

Software commands	
<b>Store data</b>	Click «Save», the file name is formed automatically from the patient's identity number and the suffix «WatchBP03_(date).xls».
<b>View the data</b>	Click <<Open excel file>>
<b>Delete the memories</b>	Click «Clear Memory»
<b>Close the program</b>	Click «Exit»



- ❖ Please unplug and re-plug the USB cable to the PC if the software instructs you to do so.
- ❖ The program will prompt to synchronize the date/time of the device to that of the PC.
- ❖ Store your data before using «Clear Memory» or «Exit».
- ❖ System Requirements: 550MHz CPU, 256MB Memory, 1024x768 pixel resolution, 256 color, CD-ROM drive, 1 free USB port, 40MB free hard disk space, Microsoft Windows XP / Vista.

# Appendix
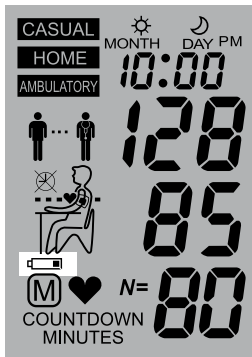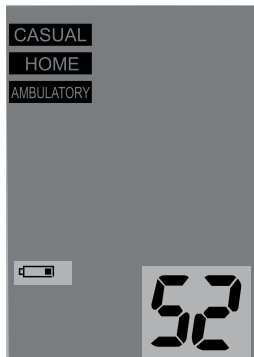
## Battery indicator

When batteries are placed into battery compartment, the battery symbol and the number indicating battery power will be displayed. Please replace new batteries for ABP patient test if the number is equal or less than 50.

When the batteries have  $\frac{1}{4}$  power supply left, the Battery Symbol will flash each time the device is switched on.


## Replacing low batteries

When the batteries need to be replaced, the Battery Symbol will flash each time the device is switched on.

- 1) Open the battery compartment at the back of the device.
- 2) Replace the batteries – ensure correct polarity as shown by the symbols in the compartment.

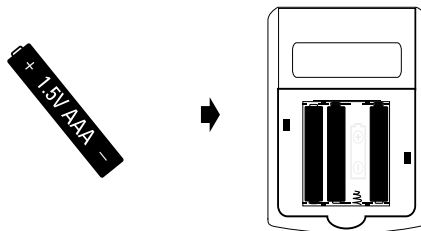- ※ Use 4 new, long-life 1.5V, size AAA batteries.
- ※ Do not use batteries beyond their date of expiry.
- ※ Remove batteries, if the device will not be used for a prolonged period.

# Safety, care, accuracy test and disposal

## Safety and protection

This device may be used only for the purpose described in this booklet. The device comprises of sensitive components and must be treated with caution. The manufacturer cannot be held liable for damage caused by incorrect application.

- Ensure that children do not use the device unsupervised; some parts are small enough to be swallowed.
- Only activate the pump when cuff is installed.
- Do not use the device if you think it is damaged or if anything appears unusual.
- Read the further safety instructions in the individual sections of the instruction manual.
- Do not connect the device to a computer until prompted to do so by the computer software.

Observe the storage and operating conditions described in the “Technical specifications” section of this manual.

**Protect the device from water and moisture**

---

**Protect the device from direct sunlight**

---

**Protect the device from extreme heat and cold**

---

**Avoid proximity to electromagnetic fields, such as those produced by mobile phones**

---

**Never open device**

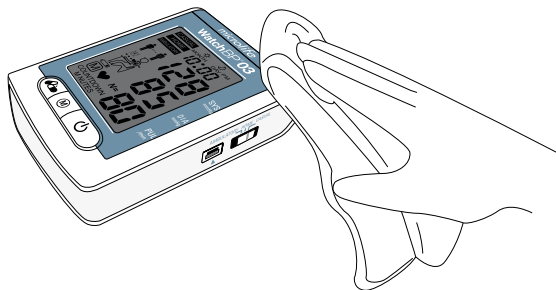
---

**Protect device from impact and drops**

## Device care

Clean the device with a soft, dry cloth.

## Accuracy test

We recommend the WatchBP 03 device be tested for accuracy every 2 years or after mechanical impact (e.g. being dropped). Please contact Microlife to arrange for an accuracy test.

## Cleaning the cuff and straps

Use a mild disinfectant solution to clean the pouch, straps, sling and belt. To wash these items and cuff in washing machine, use a cuff plug (article #90-273MZ11-004) to keep the inside of the bladder dry. Fold and place the cuff inside a washing bag. Wash these items using warm water and a mild detergent. Air dry the cuff. **DO NOT** iron the cuff cover.

**Do not iron the cuff!**

## Disposal

Batteries and electronic instruments must be disposed of in accordance with the locally applicable regulations, and not as domestic waste.

## Error messages

If an error occurs during measurement, the measurement is interrupted and an error message «Er» is displayed.

- Please consult your doctor, if this or any other problem occurs repeatedly.
- If you think the results are unusual, please read through the information in this instruction manual carefully.

Error	Description	Potential cause and remedy
«Er 1»	<b>Signal too weak</b>	The pulse signals on the cuff are too weak. Re-position the cuff and repeat the measurement.
«Er 2»	<b>Error signal</b>	During the measurement, error signals were detected by the cuff, caused for instance by movement or muscle tension. Repeat the measurement, keeping your arm still.

«Er 3»	<b>No pressure in the cuff</b>	An adequate pressure cannot be generated in the cuff. A leak may have occurred. Replace the batteries if necessary. Repeat the measurement.
«Er 5»	<b>Abnormal result</b>	The measuring signals are inaccurate and no result can therefore be displayed. Read through the checklist for performing reliable measurements and then repeat the measurement.

«HI»	<b>Pulse or cuff pressure too high</b>	The pressure in the cuff is too high (over 300 mmHg) OR the pulse is too high (over 200 beats per minute). Relax for 5 minutes and repeat the measurement.
«LO»	<b>Pulse too low</b>	The pulse is too low (less than 40 beats per minute). Repeat the measurement.

# Technical Specifications

**Operating temperature:** • 10 - 40 °C / 50 - 104 °F  
• 15 - 85 % relative maximum humidity

**Storage temperature:** • -20 - +50 °C / -4 - +122 °F  
• 15 - 90 % relative maximum humidity

**Weight:** • 260g (including batteries)

**Dimensions:** • 115 x 80 x 35 mm

**Measuring procedure:** • oscillometric, corresponding to Korotkoff

**Measurement range:** • 30 - 280 mmHg – blood pressure  
• 40 - 200 beats per minute – pulse

**Cuff pressure display:** • Range: 0 - 299 mmHg  
• Resolution: 1 mmHg  
• Static accuracy: pressure within ±3 mmHg  
• Pulse accuracy: ±5 % of the readout value

**Power source:** • 4X1.5 V Batteries; size AAA

**Reference to Standards:** • EU Directives 93/42/EEC  
• NIBP requirements:  
EN 1060-1 / -3/ -4,  
EN 60601-2-30,  
ANSI /AAMI SP10

Microlife reserves the right to alter technical specifications without prior written notice.

# Guarantee Card

This device is covered by a two-year guarantee from the date of purchase. This guarantee is valid only on presentation of the guarantee card completed by the owner confirming date of purchase or purchase receipt. Batteries and wearing parts are not covered by this guarantee.

**Name:** \_\_\_\_\_

**Address:** \_\_\_\_\_

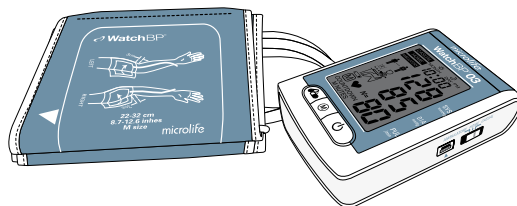
\_\_\_\_\_

**Date:** \_\_\_\_\_

**Telephone:** \_\_\_\_\_

**Email:** \_\_\_\_\_

**Product:** WatchBP 03  
**Product Number:** 3MZ1-1  
**Serial Number:**  
**Date:**

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<sup>1</sup> O'Brien E, Asmar R, Beilin L, Imai Y, et al. European Society of Hypertension recommendations for conventional, ambulatory and home blood pressure measurement. European Society of Hypertension Working Group on Blood Pressure Monitoring. J Hypertens 2003;21:821-848.

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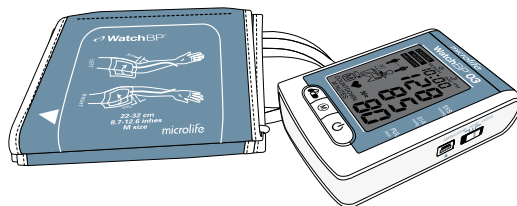
# Microlife® WatchBP O3 Ambulatory Blood Pressure Monitor

**Extended manual for explanation and  
background of blood pressure data**

## **Introduction**

Dear healthcare professional,

The present manual is offered to you by Microlife Corporation. It entails explanations of blood pressure data as provided by the Microlife® WatchBP O3 software. In addition, some scientific background is given to indicate the clinical relevance of these data. The Microlife® WatchBP O3 device is validated according to the International Protocol [1] and the reports as provided by the software are developed according to the guidelines of the European Society of Hypertension and American Heart Association. We hope that the present manual will help you for better understanding and interpretation of the blood pressure data of your patient. The recommendations in this manual do not indicate an exclusive course of treatment or serve as a standard of medical care. Variations, taking into account individual differences and circumstances, may be appropriate

Yours sincerely,

Willem Verberk, Phd  
Medical Marketing Manager  
Microlife Corporation

**Blood pressure load:** The percentage of blood pressure (BP) readings above a certain threshold

Level [2]. On default this is according to the recommended thresholds of 135/85 mmHg during daytime and 120/70 mmHg during the night time. However these threshold levels can be changed with the button “report settings”.

**Normal values:** The recommended levels of normality for ambulatory BP measurement in adults is 135 mmHg and 85 mmHg during daytime (Awake) for systolic and diastolic pressure, respectively. For night time (Asleep) these values are 120 mmHg and 70 mmHg for systolic and diastolic pressure [3]. In the PDF graph the normal values are provided with the horizontal grey bars.

**Percentage of successful readings:** According to European Society of Hypertension (ESH) recommendations the percentage of successful readings should be similar to or more than 70% [4]. When the percentage of readings is lower than 70% the number will be provided in red, which indicates that another 24-hour measurement should be taken.

**Dip:** The percentage at which blood pressure decreases at night. Dip is calculated with the following formula:  $(\text{awake BP} - \text{asleep BP}) / (\text{awake BP}) \times 100\%$  [5]. Several studies have shown that there is an inverse correlation with cardiovascular risk and blood pressure decrease during the night [6, 7]. Some even have shown that the nocturnal blood pressure is the best prognostic factor for cardiovascular mortality [8, 9]. Patients can be classified as dippers ( $\geq 10\%$  decrease in BP at night) and non-dippers ( $< 10\%$  decrease in BP at night) the latter group is expected to have a poorer cardiovascular prognosis than dippers [6]. See also Figure 1 for classification based on calculation.

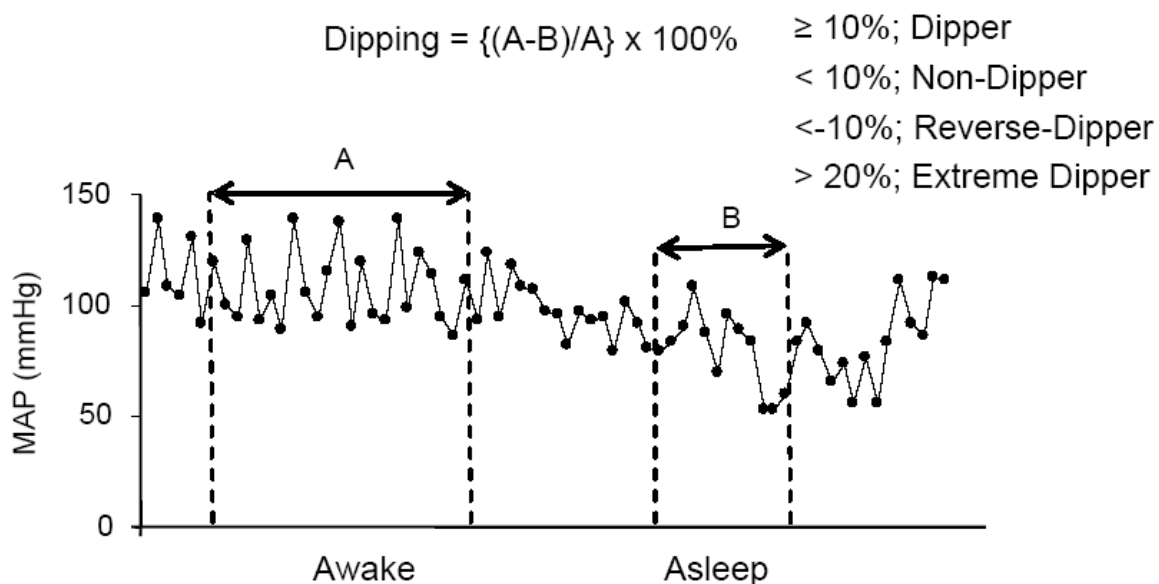
Figure 1.

**Hourly average:** The average BP values are calculated from the average BP value of each hour separately during 24 hour. This method is chosen to give similar weight to every hour of the day. In practice this means i.e. for the average value of 24h BP measurement a nocturnal hour is just as important as any hour during the day despite the fact that during daytime BP probably was measured more frequently.

**Standard deviation (SD):** This index indicates the absolute variability of the BP or pulse of a given period. It is calculated with the following formula:

$$\sigma = \sqrt{\frac{1}{N} \sum_{i=1}^N (x_i - \mu)^2},$$

$\sigma$  = Standard deviation

$N$  = The number of BP (or pulse) values

$\mu$  = The average BP (or pulse) value

$\Sigma$  = The total sum

It has been shown that the BP variability correlates closely with target-organ damage and with the incidence of cardiovascular events, independent of absolute BP levels [10].

**24-hour:** The average BP value calculated from the total number of BP values.

**Awake:** The average BP values calculated from BP values as measured when the patient was awake (daytime BP)\*.

**Asleep:** The average BP value calculated from all values as measured when the patient was sleeping (night time BP)\*.

\*Before each measurement the day and night period can be set with the button "Ambulatory Settings". However, when these time points deviate from those as given by the patient afterwards, the actual awake and asleep period can be programmed with the button "Report settings". The average awake and asleep values will then be calculated on the basis of these actual time points as provided by the patient.

**Pulse:** Heart rate provided in beats per minute.

**Mean arterial pressure (MAP):** Defined as the average arterial pressure during a single cardiac cycle. Oscillometric devices automatically measure the MAP and calculate the systolic and diastolic pressure from there. For calculating the MAP from systolic and diastolic BP the formula is as follows:

$$\text{MAP} \approx \text{DP} + \frac{1}{3} (\text{SP}-\text{DP})$$

DP = diastolic pressure

SP = systolic pressure

**Pulse Pressure (PP):** Calculated with the formula:

$$\text{PP}=\text{SP}-\text{DP}$$

SP = systolic pressure


DP = diastolic pressure

PP is a measure of the stiffness of the aorta and large arteries. Increased stiffness leads to an increase in PP through a reduction in arterial compliance and effects on wave reflection [11] PP has been recognized as an independent predictor of cardiovascular risk in a hypertensive [12] and general population [13].

**White coat window:** The maximum reading in the first hour of ambulatory BP measurement. As in this first hour the patient normally is in or has just left the hospital or GP's practice, the BP values largely reflect the office BP measurement. The white coat window has proven its clinical value for detection of the white coat effect and/or white coat hypertension [14].

**Diary:** Highly recommended to any patient who undergoes 24h ambulatory BP measurement. The actual awake and asleep periods should be based on this diary. With the diary deviating BP values could be explained (i.e. patient has just had his meal, held a siesta, was smoking or was watching a football match) and the time of (antihypertensive) drug intake should be registered, which allows verification of the effectiveness of the drug. For the latter, the WatchBP O3 device is equipped with a button that should be pressed by the patient directly after antihypertensive drug intake.

**Indication for ambulatory BP measurement:** the recommended procedure about when to perform 24h BP measurements is provided in Figure 2. The Table provides indications at which ambulatory BP measurement should and could be considered.

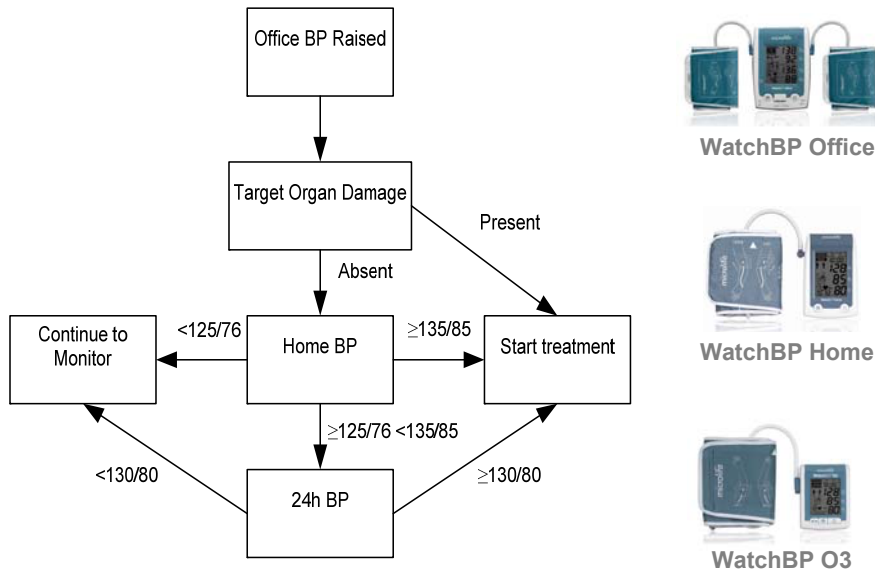**Figure 2.** Schema for evaluating BP status of hypertensive patients [15]

**Table.** Indication for performing 24h blood pressure measurement

Accepted indications	<ul style="list-style-type: none"> <li>Suspected white-coat hypertension</li> <li>Suspected nocturnal hypertension</li> <li>Suspected masked hypertension</li> <li>To establish dipper status</li> <li>Resistant hypertension</li> <li>Hypertension of pregnancy</li> </ul>
Potential indications	<ul style="list-style-type: none"> <li>Elderly patient</li> <li>As a guide to antihypertensive drug treatment</li> <li>Type 1 diabetes</li> <li>Evaluation of symptoms suggesting</li> <li>Orthostatic hypotension</li> <li>Autonomic failure</li> </ul>

## References

1. O'Brien E, Pickering T, Asmar R, Myers M, Parati G, Staessen J *et al.* Working Group on Blood Pressure Monitoring of the European Society of Hypertension International Protocol for validation of blood pressure measuring devices in adults. *Blood Press Monit* 2002; **7**: 3-17.
2. Mule G, Nardi E, Andronico G, Cottone S, Raspanti F, Piazza G *et al.* Relationships between 24 h blood pressure load and target organ damage in patients with mild-to-moderate essential hypertension. *Blood Press Monit* 2001; **6**: 115-123.
3. O'Brien E, Asmar R, Beilin L, Imai Y, Mallion JM, Mancia G *et al.* European Society of Hypertension recommendations for conventional, ambulatory and home blood pressure measurement. *J Hypertens* 2003; **21**: 821-848.
4. Mancia G, De Backer G, Dominiczak A, Cifkova R, Fagard R, Germano G *et al.* 2007 Guidelines for the Management of Arterial Hypertension: The Task Force for the Management of Arterial Hypertension of the European Society of Hypertension (ESH) and of the European Society of Cardiology (ESC). *J Hypertens* 2007; **25**: 1105-1187.
5. Verdecchia P, Schillaci G, Guerrieri M, Gatteschi C, Benemio G, Boldrini F *et al.* Circadian blood pressure changes and left ventricular hypertrophy in essential hypertension. *Circulation* 1990; **81**: 528-536.
6. O'Brien E, Sheridan J, and O'Malley K Dippers and non-dippers. *Lancet* 1988; **2**: 397.
7. Staessen JA, Thijs L, Fagard R, O'Brien ET, Clement D, de Leeuw PW *et al.* Predicting cardiovascular risk using conventional vs ambulatory blood pressure in older patients with systolic hypertension. Systolic Hypertension in Europe Trial Investigators. *JAMA* 1999; **282**: 539-546.
8. Dolan E, Stanton A, Thijs L, Hinedi K, Atkins N, McClory S *et al.* Superiority of ambulatory over clinic blood pressure measurement in predicting mortality: the Dublin outcome study. *Hypertension* 2005; **46**: 156-161.
9. Sega R, Facchetti R, Bombelli M, Cesana G, Corrao G, Grassi G *et al.* Prognostic value of ambulatory and home blood pressures compared with office blood pressure in the general population: follow-up results from the Pressioni Arteriose Monitorate e Loro Associazioni (PAMELA) study. *Circulation* 2005; **111**: 1777-1783.
10. Parati G, Faini A, and Valentini M Blood pressure variability: its measurement and significance in hypertension. *Curr Hypertens Rep* 2006; **8**: 199-204.
11. Dart AM and Kingwell BA Pulse pressure--a review of mechanisms and clinical relevance. *J Am Coll Cardiol* 2001; **37**: 975-984.
12. Asmar R, Rudnichi A, Blacher J, London GM, and Safar ME Pulse pressure and aortic pulse wave are markers of cardiovascular risk in hypertensive populations. *Am J Hypertens* 2001; **14**: 91-97.
13. Benetos A, Safar M, Rudnichi A, Smulyan H, Richard JL, Ducimetieere P *et al.* Pulse pressure: a predictor of long-term cardiovascular mortality in a French male population. *Hypertension* 1997; **30**: 1410-1415.
14. Owens P, Atkins N, and O'Brien E Diagnosis of white coat hypertension by ambulatory blood pressure monitoring. *Hypertension* 1999; **34**: 267-272.
15. Pickering TG, Miller NH, Ogedegbe G, Krakoff LR, Artinian NT, and Goff D Call to action on use and reimbursement for home blood pressure monitoring: a joint scientific statement from the American Heart Association, American Society Of Hypertension, and Preventive Cardiovascular Nurses Association. *Hypertension* 2008; **52**: 10-29.

### Recommended literature

- I O'Brien E, Asmar R, Beilin L, Imai Y, Mancia G, Mengden T et al. Practice guidelines of the European Society of Hypertension for clinic, ambulatory and self blood pressure measurement. *J Hypertens* 2005; **23**: 697-701.
- II Pickering TG, Shimbo D, and Haas D Ambulatory blood-pressure monitoring. *N Engl J Med* 2006; **354**: 2368-2374.
- III Segà R, Facchetti R, Bombelli M, Cesana G, Corrao G, Grassi G et al. Prognostic value of ambulatory and home blood pressures compared with office blood pressure in the general population: follow-up results from the Pressioni Arteriose Monitorate e Loro Associazioni (PAMELA) study. *Circulation* 2005; **111**: 1777-1783.

### Recommended websites

[www.watchbp.com](http://www.watchbp.com) (also for downloading the newest WatchBP software)  
[www.dableducational.org](http://www.dableducational.org)  
[www.bhsoc.org](http://www.bhsoc.org)

When you have more questions about the context of this manual or need to know more about blood pressure data that are not mentioned in the present manual, please contact the following email address:

[willem.verberk@microlife.com.tw](mailto:willem.verberk@microlife.com.tw)